

4. Adjusting the Webbing



5. Spare Parts:



Promobility Ltd.

Unit A3, Airside Enterprise Centre,
Airside Business Park, Swords,
Co. Dublin, K67 YY94, Ireland.

Tel: +353 (1) 8900890

Email: sales@promobility.ie www.promobility.ie



Promobility Magnet Harness

User Guide



It is important to read and understand this guide before installation and use of the Magnet Harness, this will help ensure its optimal use and performance.

The Magnet Harness is designed for use as a postural support harness in a vehicle, car, bus, etc. and is NOT a safety belt, you MUST ensure that the Magnet Harness is used only in conjunction with the vehicles seat belt system for the user's safety.

The Magnet Harness is designed for use by seated passengers and there are a variety of sizes available to suit children up to adult.

The Magnet Harness is NOT a safety belt, but a fixation belt to ensure the user's proper location on the vehicle seat.

The Magnet Harness must not be used for any purpose other than described, the CE marking and warranty are only valid with correct use.

1. How to Use
2. Maintenance
3. Warranty
4. Adjusting the Webbing
5. Spare Parts



1. How to Use:



Place the harness pad against the seat backrest, place shoulder straps with triangular assembly plate to the rear of the seat backrest, feed these below the headrest of the seat. Place the lower triangular assembly plate with the lap and crotch straps through the gap at the bottom of the backrest.



Position the triangular assembly plate high on the back of the backrest, close the buckle and adjust both shoulder belts. This is done by tightening the belt on the triangular assembly plate.

Adjust the lap and crotch straps as necessary.

With the magnet key, remove the magnetic buttons on the harness pad and crotch strap assembly.

Place the passenger on the seat making sure the crotch belt is brought through. Set crotch belt loosely around the person.

Close the harness on the passenger, placing a magnet button on the top fixing, then close the crotch strap in position, placing both lower buttons.

The crotch belt should not be too tight.

Removing the magnet buttons is done by placing the red magnet key flat on the magnet buttons. You may need to pinch together the layers of harness material, particularly at the crotch strap, so that the magnet button is free to spin, this will make their removal simple.

Make sure the chest pad is positioned at sternum height and is not allowed to ride upward toward the neck.



Make sure the chest pad is positioned at sternum height and is not allowed to ride upward toward the neck.

The passenger is now secure in the seat. make adjustments to the Lap, Shoulder and Crotch straps as necessary.

- The Lap strap should be comfortably tight.
- The Crotch strap should have no slack but for comfort do not overtighten.
- Shoulder straps should be adjusted so as to allow space to place your flat hand inside the straps.
- Ensure the chest panel is not allowed to ride up on the passenger, keep appropriate clearance from the passengers neck, as shown in the picture opposite.
- Your harness is not a safety belt, Always use the vehicle seatbelt.

2. Maintenance:

The Magnet Vest Harness can be washed at 40°C, do not tumble dry.

NOTE: Do not wash the Magnet Buttons as damage will occur.

Daily maintenance before use:

Inspect the harness for damage and wear.

Check that all magnet buttons function correctly using the magnet keys provided. Ensure fixing strap buckles are intact and in working order, including webbing adjusters. If damage or wear are present, remove the harness from service and arrange for repair or replacement.

3. Warranty:

The manufacturer's warranty covers manufacturing, materials and functions of the equipment supplied. The warranty is applicable when the equipment is operated in accordance with instructions for use and for the purpose for which the equipment is designed.

The warranty covers repair of defects in the equipment supplied.

Warranty does not cover misuse, abuse, or damages due to misuse and abuse.

All repairs and replacement of components during the warranty period shall be made only with prior consent and approval of the manufacturer.

Warranty does not cover shipping, postage, packaging or any other costs associated.

The warranty period is for 12 months from date of supply.

Never leave passengers unattended, when using the Magnet Harness or any other harness system.